# geemarc™ L2T-10

Hear the Difference

## Loop to Text convertor

En

!!! Please check our website: [www.geemarc.com](http://www.geemarc.com) for the latest user manual as there may be important updates and changes you need to be aware of !!!

Congratulations on your purchase of the Geemarc™ L2T-10. This is a portable induction loop receiver system that can be connected to a variety of devices that have a USB type C connector.

This system allows you to pick up electromagnetic fields in cinemas, ticket offices or museums that are equipped with electromagnetic field transmission systems.

You can connect your L2T-10 to your mobile phone or tablet and then use a transcription application to read the information on your device's screen.


The range of your system is approximately 3 to 5 metres maximum, depending on the strength of the transmitted signal.

### Unpacking the L2T-10

When unpacking the L2T-10, you should find the following in the box:

- 1 Geemarc™ L2T-10
- 1 Quick user guide

### General Description of your L2T-10

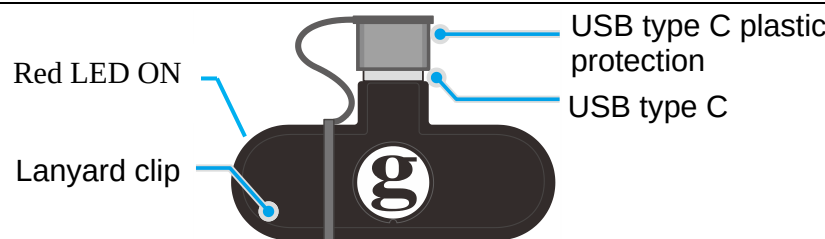### Using the L2T-10 with your Smart-Phone or Tablet?

- 1) In first, remove the plastic protection before to connect the USB type C in your smartphone or tablet.
- 2) Download the Live Transcribe app from Google Play for example Appl free on google play.
- 3) Connect the L2T-10 to your smartphone or tablet as shown in the diagram.

- 4) Run the Live Transcribe APP on your smartphone or tablet by tapping twice on the application icon you downloaded.

When you receive the electromagnetic signal, the speech is automatically transcribed to text in Live Transcribe APP.

- 5) Your product is ready to use.

### Troubleshooting

#### The L2T-10 does not seem to work

- If the red LED doesn't light up, invert the L2T10 in the USB type C port and check that the OTG function is enabled in your parameters. Check that the red light is on.
- The Live Transcribe application does not seem to work properly. Ensure that the L2T-10 is securely connected to the USB Type C connector on the smartphone or tablet.
- Check that the signal received is correct in the top right corner of the application.
- Go to SETTINGS then select more settings, make sure the selected

microphone is set to "USB Audio Geemarc Loop".

- Make sure you are speaking clearly into the phone.
- Make sure the microphone information in the APP settings under Audio & Speech doesn't say "this device" but "USB Audio Geemarc Loop".

**IMPORTANT:** You can find video on our Website to help you.

Go to on web side « [www.geemarc.com/uk/L2T10/product\\_files](http://www.geemarc.com/uk/L2T10/product_files) »

### Guarantee

Your Geemarc™ L2T-10 is guaranteed by Geemarc™ for two years from the date of purchase. During this period, all repairs or replacements (at our discretion) will be free of charge. Should you experience a problem, please contact our Helpline or visit our website at [www.geemarc.com](http://www.geemarc.com). The warranty does not cover accidents, negligence or breakage of any part. The product must not be tampered with or dismantled by anyone other than an authorised Geemarc™ representative. The Geemarc™ warranty does not limit your statutory rights in any way.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR WARRANTY AND MUST BE RETAINED AND PRESENTED IN THE EVENT OF A WARRANTY CLAIM.

### DECLARATIONS:

Geemarc™ Telecom SA hereby declares that this L2T10 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014 / 53 / UE.

The UKCA or CE Declaration of Conformity can be viewed at [www.geemarc.com](http://www.geemarc.com).

**FC** This device complies with Part 15 of the FCC Rules [and with RSS-210 of Industry Canada].

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference AND
- (2) This device must accept any interference received, including interference that may cause undesired operation.

The WEEE (Waste Electrical and Electronic Equipment) Directive has been introduced to ensure that products are recycled in the best possible way at the end of their useful life.

When you are finished with this product, please do not dispose of it in your household waste bin.

Please dispose of it in one of the following ways:

- Remove the batteries and dispose of them in an appropriate WEEE container. Dispose of the product in an appropriate WEEE container.
- Or return the old product to your retailer. If you buy a new one, they should accept it.

By following these instructions, you will help protect human health and the environment.

For product support and help, visit our website at [www.geemarc.com](http://www.geemarc.com)

E-mail: [help@geemarc.com](mailto:help@geemarc.com)

Telephone: 01707 387602

Lines are open from 10:00-12:00 then 13:00-14:30 Monday to Friday

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Made for Geemarc Telecom S.A. in China

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Hergestellt für Geemarc Telecom S.A. in China