

## Hearing Mode Selection

- Press and hold the function button for more than 3 seconds. The white LED will turn on, followed by a colored LED indicating the current mode.
- Use the UP and DOWN buttons to cycle through the 7 available hearing modes.
- To save your selected mode, press the function button for short. The LED will blink three times to confirm.



## Bluetooth Earphone Connection

- When the device is powered on, the red Bluetooth LED blinks intermittently
- If you open the earphone case at this time, earphones auto-connect and the red LED turns off.

## Master Volume Control

- Press the UP button. The white LED turns on and the volume increases.
- Press the DOWN button. The white LED turns on and the volume decreases.

● Minimum (1) ○ Medium (2-15) ● Maximum (16)

## Independent Volume Control

- Press and hold the UP button to enter right-side volume control mode. Press and hold the DOWN button to enter left-side volume control mode.
- When mode is activated, the yellow LED lights up. Use the UP, DOWN buttons to adjust the volume.
- To save your settings, press the function button. The yellow LED turns off, and the green LED blinks twice to confirm.

## Mute Function

- Press the function button to mute the microphone and the yellow LED will turn on.
- Press it again to unmute, and the yellow LED will turn off.

## Using External Input


- Connect an audio source to the USB-C port. Hold the UP button and power on the device to hear the audio through your earphones.



Bluetooth® LE Audio



GENIESORI

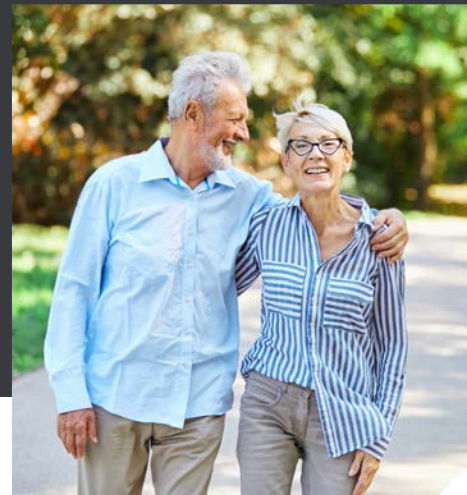

Premium Assistive Listening Device


# GENIESORI3 (GLE)

## User Guide

Experience crystal clear sound through the fusion of hearing aid technology and next-gen wireless audio.

# GENIESORI III

## Product Overview

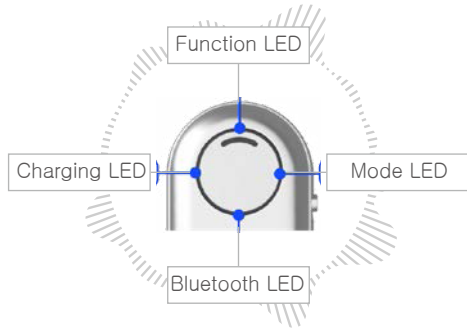

UWICOM  
Your Wish Is My Command

## Power On & Off

- Slide the power switch to ON to turn on the device. The function LED will blink three times to indicate the battery level.
- To turn off the device, slide the switch to OFF.



● Low battery    ● Medium level    ● Fully charged

## Battery Charging

- Plug the charger into the USB Type-C port.
- The red Charging Status LED lights up during charging and turns green when fully charged.



## Connecting New Earphones

Use this function to disconnect current earphones and connect new ones.

- ① Press and hold the function button for more than 10 seconds, and LED will change from white to red.
- ② While the red LED is on, turn off the power. The current earphone connection will be removed.
- ③ Set your new earphones to pairing mode.
- ④ Turn on GENIESORI 3 again. It will automatically connect to the new earphones.

! This product supports Bluetooth LE Audio only. Please make sure your earphones support LE Audio.

## Product Information

- Certified: KC, CE
- Dimensions: W 28.2 × H 82 × T 12.2 (mm)
- Weight: 27g
- Bluetooth Version: 5.3
- Battery Capacity: 3.7V 550mAh
- Operating Temperature: -10 to 50°C

## Product Information

- Prolonged exposure to loud sounds may cause hearing damage.
- Do not drop or disassemble the product.
- Keep the device away from water, moisture, and extreme temperatures.
- Risk of explosion if the battery is replaced incorrectly. Follow proper disposal and replacement steps.

## UWICOM Co., Ltd.

(16976) 58, Giheung-ro, Giheung-gu, Yongin-si  
Gyeonggi-do, Republic of Korea  
TEL. +82-31-256-4427 FAX. +82-504-216-1754  
E-mail. [info@uwicom.com](mailto:info@uwicom.com)  
Website. [www.uwicom.com](http://www.uwicom.com)

