HUMANTECHNIK

VC-2

DE

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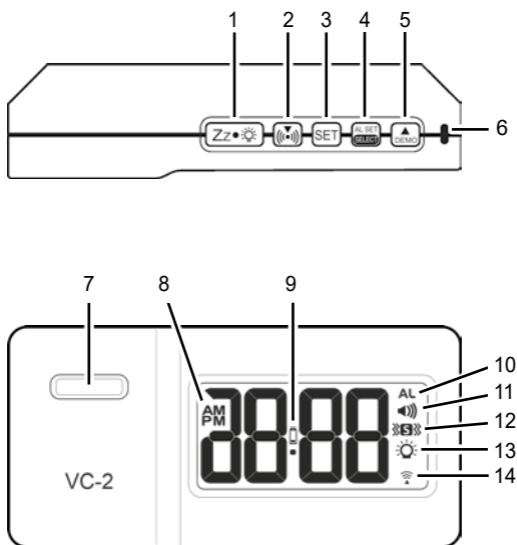
12 Vibration active

14 RF symbol

Thank you for purchasing the »VC-2«! You have chosen a modern and reliable device.

Intended use

The »VC-2« is an electronic clock with digital time display and automatic time synchronisation. It wakes you up with an acoustic signal, vibrations and/or flashing lights.

Scope of delivery:

- »VC-2« vibrating alarm clock
- 3 AA batteries (LR6)
- Carrying strap with clip
- Storage bag
- Operating instructions and warranty card

Before using the device

- Please read the operating instructions carefully. This will familiarise you with your new device and teach you about all its functions and components. You will learn important details about starting up and using the device.
- Following the operating instructions will prevent damage to the device and avoid jeopardising your statutory rights in the event of misuse.
- We are not liable for incorrect measurements and any consequences that may result from them.
- Humantechnik GmbH accepts no liability for damage caused by failure to observe these operating instructions.
- Pay particular attention to the safety instructions! Keep these operating instructions in a safe place!

For your safety

• The product is designed exclusively as a vibrating alarm clock. Do not use the product for any other purpose.

Caution! Risk of injury:

- Keep the device and batteries out of the reach of children.
- Small parts may be swallowed by children under three years of age.
- Batteries contain harmful acids and can be life-threatening if swallowed. Swallowing a battery can cause severe internal burns and death within 2 hours.

If you suspect that a battery has been swallowed or has otherwise entered the body, seek medical attention immediately.

- Do not throw batteries into fire, short-circuit them, take them apart or recharge them. Risk of explosion!
 - To prevent batteries from leaking, replace weak batteries as soon as possible.
- Never use old and new batteries or batteries of different types at the same time.

- Ensure correct polarity when inserting batteries. Only use LR6 (AA) batteries.
 - Remove the batteries if you are not going to use the device for a long period of time (>2 months).
 - If a battery leaks, avoid contact with skin, eyes and mucous membranes.
- If necessary, rinse the affected areas immediately with water and consult a doctor.

Product safety information

Please observe these instructions to avoid damage to the device and, if applicable, voiding the warranty.

- The device is designed for use in residential areas. The device must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical vibrations.
- Only place the device on flat surfaces. Do not drop the device.

If the device falls onto a hard surface, it may be damaged or destroyed.

- The device must not be dismantled. Opening or modifying the device will result in the immediate expiry of the warranty.
- This device is not suitable for outdoor use. The device must not be exposed to moisture or water.

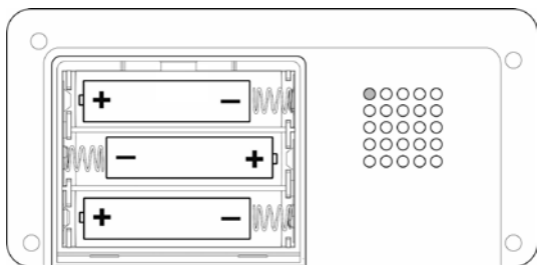
Getting started

1. Inserting the batteries

The »VC-2« requires 3 AA (LR6) batteries. Open the battery compartment cover by sliding it downwards.

Insert the 3 supplied batteries into the battery compartment according to the markings. Observe the specified polarity.

Then replace the battery compartment cover.

Please note:

The more frequently the vibration alarm, light or loud alarm tone are used, the faster the batteries will run out.

When the battery level is low, the battery  symbol appears between the digits.

In this case, replace the batteries as soon as possible.

Please dispose of used batteries in accordance with legal regulations.

Time signal reception

The »VC-2« synchronises the exact time via the DCF77 radio signal. After inserting the batteries, it starts receiving the radio time signal. The RF symbol  (14) flashes.

If the time synchronisation is successful, the symbol is displayed permanently. If reception is faulty, the symbol disappears.

Notes on radio reception

As with all radio and wireless devices, reception may be affected by the following factors:

- Too great a distance from the transmitter (see below)
- Nearby buildings, mountains and forests
- Nearby power lines and construction sites
- Position inside solid concrete buildings
- Nearby devices such as computers and monitors
- Position inside vehicles
- Large metal objects in the vicinity.

For optimal reception of the time signal, place the »VC-2« near a window away from metal objects and electrical devices.

The DCF77 time signal transmitter

The DCF77 is a time signal transmitter that broadcasts the current date and time in digital form on the longwave frequency 77.5 kHz.

The time is CET (Central European Time) or CEST (Central European Summer Time).

The DCF77 transmitter is located in Mainflingen in Hesse, Germany. It has a range of approximately 2000 km.

Automatic reception

Your alarm clock starts automatic reception every day at 01:00 AM. If the signal is not received, time synchronisation will be attempted again every hour until 05:00 AM. If this fails repeatedly, reception will start again at 01:00 AM the next day.

With the exception of the snooze/backlight button **Zz** ● ☀ (1), none of the buttons will function during reception.

Manual reception start

Press and hold the RF/DOWN button (📶) (2). The time signal reception starts with a short beep. To stop reception, press and hold this button again.

Manual time setting

If the time cannot be synchronised via radio reception, it can also be set manually:

1. Press and hold the **SET** button (3) until the display flashes »00«. Press the UP and DOWN buttons (2) and (5) marked ▲/▼ to set the time zone (up to ±12 hours).

This setting also affects the radio time synchronisation!

»00« = GMT +1 hour (Germany, France, Italy)

»01« = GMT +2 hours (e.g. Finland, Eastern Europe)

»-01« = GMT 00 hours (Great Britain)

2. Press the **SET** button again. The hour digits will start flashing. Press the ▲ / ▼ buttons to set the hours (hold down the buttons to scroll quickly).

3. Press the **SET** button again to set the minutes. To do this, press the up and down buttons ▲ / ▼ again (hold down for fast scrolling).

4. Press the **SET** button again and the time format will flash on the display. Press the **▲ / ▼** buttons to select the time format »12Hr« or »24Hr«. If »12Hr« is selected, »AM/PM« (8) will appear on the display.

5. Press the **SET** button again. »05« will flash on the display. Press the **▲ / ▼** buttons to select your snooze time from 5 to 60 minutes.

6. Press the **SET** button again to exit the setting mode.

If no button is pressed within approximately 10 seconds, the setting mode will automatically exit.

Setting the alarm time

1. Press and hold the **AL SET/SELECT** button  (4). The **AL** indicator (10) appears and the hour digits of the alarm time flash.

Press the **▲ / ▼** buttons to set the hours of the desired alarm time (press and hold the buttons for fast scrolling).

2. Press the **AL SET/SELECT** button again to switch to the minute setting.

Press **▲** or **▼** to set the minutes of the alarm time (press and hold the buttons for fast scrolling).

3. Press the **AL SET/SELECT** button again.

The **»S«** indicator flashes and »HI« is displayed (high vibration intensity). Press **▲** or **▼** to select the vibration intensity:

»HI« = high vibration intensity

»LO« = low vibration intensity

4. Press the **AL SET/SELECT** button again.

The  indicator flashes and the display shows »HI«. Press  or  to set the alarm volume:

»LO« = lower alarm tone volume

»HI« = higher alarm tone volume

5. Press the **AL SET/SELECT** button again to exit the setting mode.

If no button is pressed within approximately 10 seconds, the setting mode will automatically exit.

Set wake-up method

1. In normal time display mode, press the **AL SET/SELECT** button to set the alarm method.

The corresponding symbols on the display will be indicated:

 : Vibration

 : Tone and vibration

 : Flashing light and vibration

  : Flashing light, tone and vibration

 : Tone and flashing light.

2. Press the **AL SET/SELECT** button again to switch off all alarm methods. The indicators ,  and  are off.

Stopping the alarm

When the alarm sounds, the corresponding alarm method symbols flash. Press the **AL SET/SELECT** button once to stop the alarm until the next day. The corresponding alarm method symbols remain lit on the display.

Snooze function

When the alarm is given, press the snooze/backlight button **Zz** ● ☀ .

The alarm will be temporarily turned off, but the corresponding alarm method indicators will continue to flash. The alarm will sound again after the set snooze time has elapsed.

To turn off the alarm completely, press the **AL SET/SELECT** button.

Display illumination

In normal time-display mode, briefly press the snooze/illumination button to switch on the display illumination for 5 seconds. This function is deactivated when the wake-up alarm is given.

Demo mode

Press and hold the **▲/DEMO**-button for about 2 seconds. Sound, vibration and flashing light are triggered.

Torch function

In normal time-display mode, press and hold the snooze/illumination button to turn on the torch.

To turn it off, press and hold the snooze/backlight button again.

Low battery indicator

When the battery is low, the vibration intensity, alarm tone volume and brightness are reduced.

The  symbol appears between the time digits. Replace the batteries as soon as possible.

Specifications

Dimensions (WxHxD):	128 x 63 x 32 mm
Weight (without batteries):	124 g
Power supply:	3 x AA batteries (LR6)
RCC frequency:	77.5 kHz
Alarm duration:	1 minute
Snooze time:	5 to 60 minutes

Maintenance and cleaning

- The device is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants.
- Never use acetone or hydrogen peroxide-based cleaning agents or disinfectants, or any bleaching, corrosive, or abrasive cleaning agents. No liquid or moisture may enter the openings of the device.

Disposal

This product and its packaging can be recycled and re-used.

This reduces waste and protects the environment. Dispose of the packaging in an environmentally friendly manner using the established collection systems.

Disposal of the electrical appliance

Remove the batteries from the appliance and dispose of them separately. This appliance is marked in accordance with the EU Directive on the disposal of waste electrical and electronic equipment (WEEE).

This product must not be disposed of with household waste. The user is obliged to return the old device to a designated collection point for the disposal of electrical and electronic equipment for environmentally friendly disposal. Return is free of charge. Please observe the currently applicable regulations!

Disposal of batteries

Batteries and rechargeable batteries must not be disposed of with household waste! They contain harmful substances that can damage the environment and health if disposed of improperly.

As a consumer, you are legally obliged to return used batteries and rechargeable batteries to retailers or appropriate collection points in accordance with national or local regulations for environmentally friendly disposal. Return is free of charge.

Service, warranty and warranty conditions

The device is extremely reliable. Should faults occur despite correct operation, please contact your specialist dealer or contact Humantechnik GmbH directly.

The warranty includes free repair and free return despatch. If possible, please return the devices in their original packaging. We recommend keeping the packaging for the duration of the warranty period.

The warranty is void in the event of damage caused by improper handling or repair attempts by unauthorised persons (destruction of the device seal).

Damage caused by forces of nature, such as fire and

flooding, is also not covered by the warranty.
Warranty repairs will only be carried out on submission of the completed warranty card and a copy of the invoice/receipt from the dealer.

The serial number must always be indicated.

This device fulfils the requirements of the following EU directives:

- 2011 / 65 / EU RoHS Directive
- 2012 / 19 / EU WEEE Directive
- 2014 / 53 / EU RED Directive

Conformity with the above directives is confirmed by the CE mark on the device.

CE compliance declarations are available on the Internet at www.humanteknik.com/service.

Humanteknik's UK authorised representative:

Sarabec Ltd. - 15 High Force Road
Middlesbrough TS2 1RH - United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.

UK declaration of conformity available from: Sarabec Ltd.

All given information is subject to change without prior notice.

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Humantechnik Finland Oy