



Features Defined

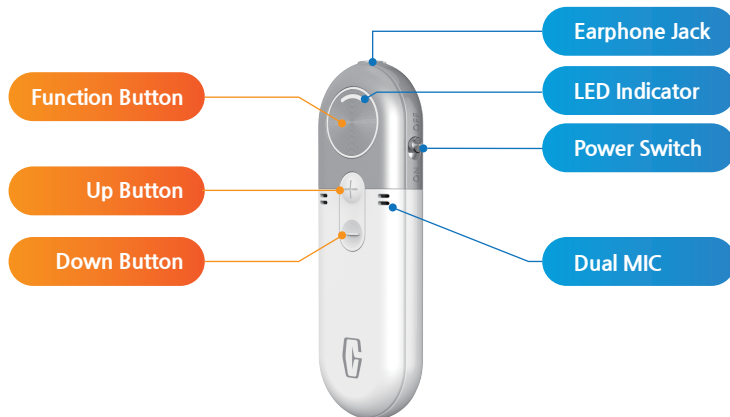

Default usage method


Power On and Off.

At power-on, the LED on the selected tone flashes 1 time and the **green LED** flashes 2 times

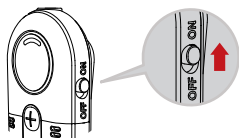

Battery status and charging.

Notifies the battery condition when power is turned on. Charge with a Micro USB Cable. During charging, the **red LED** lights up and it goes off when charging is completed.

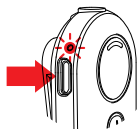

Basic operation of buttons.

Divide into short and long presses depending on the time you press.

- Short : When the button is pressed and hold within 2 sec. (the white LED is on)
- Long : When the button is pressed and hold for more than 2 sec. (the **blue LED** is on)



Volume control.

You can adjust the sound volume in 16 steps (LED shown) with a short press of the "+" up button and the "-" down button.

- Low volume (1) : Red LED
- Medium volume (2-15) : Green LED
- High volume (16) : Blue LED

Microphone Mode

- Amplifies the microphone sound according to the selected DB.
- Select the hearing-mode.

At first, press the "Down Button" for long to enter DB selection mode. And then, press the "Function Button" for short. Every time you press it, you can hear and select 7 different tones. After you select mode, press the "Down Button" long again to save tone.

* If you want to change the mode later, follow this guide.



- Press the Function button for short when you want to enter the Noise Cancellation (NC) mode.

- Change to Bluetooth mode.

Press the function button long to change to the Bluetooth mode.



- Connect a phone call.

It must be registered to the smartphone through pairing. When a call is incoming, press the function button long to connect the voice call automatically.




Bluetooth mode

- Amplifies bluetooth sound according to the selected DB.
- How to pair a Bluetooth device?

Press the "+" up button long to enter pairing mode.
(The green and red LEDs flash repeatedly)

Start searching for Bluetooth in the settings on your smartphone,
then connect the "GENIESOR2 PSA" device.

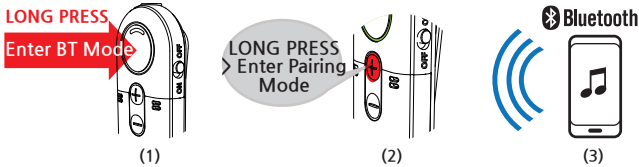

- Listening to music or voice call of the Bluetooth device.

Press the function button shortly to use it.



- Change to microphone mode.

Press the function button long to change to the microphone mode.



- How to listen to SVS-T.

The SVS-T broadcast can be received by pressing the function button and turning on the power switch.



To prevent possible hearing damage when using a headset,
do not listen to sound at high volume for long periods.

CAUTION

RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING TO THE INSTRUCTIONS

Size / Weight	W 28.2 X H 82 X T12.2 [mm], 27g
Bluetooth / Usable frequency / Modulation	4.2 / 2,402 ~ 2,480 MHz / FHSS
Continuous usage time	Max 9 hour / 3.7V 450mAh / -10 ~50°C
Battery capacity / Service temperature	



FCC certification number
HA1000 : 2AS7E-SVS-R



Personal Sound Amplifier (PSA)

HA1000 User's Guide

Ver 2.2

58, Gihneung-ro, Gihneung-gu, Yongin-si, Gyeonggi-do, 16976, South Korea
Tel. +82-31-265-4427 FAX. +82-504-373-4428
www.uwicom.com E-mail. info@uwicom.com

GENIESOR II



Bluetooth
Venture for Tomorrow
Made in Korea
RoHS
CE
FCC